



Neocera™

Advanced Cream

For Topical Dermatological Use Only



Neocera™

Advanced Cream

For Topical Dermatological Use Only



Exeltis

Rethinking healthcare

NEOCERA™ CREAM

Advanced Cream

For Topical Dermatological Use Only

Prescription Medical Device – Caution: Federal Law restricts this device to sale by, or on the order of, a licensed healthcare practitioner.

DESCRIPTION

Neocera™ Advanced Cream is a fragrance-free, water soluble dressing formulated for the management of various types of dermatoses, including atopic dermatitis and allergic contact dermatitis.

INGREDIENTS

Carbomer, dimethicone, ethylparaben, glycerin, glycyrrhethinic acid, hydroxypropyl bispalmitamide MEA (ceramides), methylparaben, palmitamide MEA, petrolatum, phenoxyethanol, polysorbate 20, propylene glycol, propylparaben, purified water, sodium hydroxide, stearic acid, trolamine, vitis vinifera (grape) seed extract.

INDICATIONS FOR USE

Neocera Cream is indicated for management of various types of dermatoses, including atopic dermatitis and allergic contact dermatitis.

CONTRAINDICATIONS

Known hypersensitivity to any of the Neocera Cream ingredients.

PRECAUTIONS:

General

Neocera Cream is to be used only as directed by a healthcare practitioner. Applicable for less than 24 hours only. It should not be used to treat any condition other than that for which it is prescribed. **For external use only.** Avoid contact with the eyes, lips, and other mucous membranes. Exposure of the eye to Neocera Cream may result in reactions such as stinging and ocular irritation.

If a reaction to Neocera Cream suggesting sensitivity or chemical irritation occurs, use of the medication should be discontinued and the prescribing healthcare practitioner consulted. This product provides no sunscreen protection. Use of sunscreen products and protective clothing over treated areas is recommended when exposure cannot be avoided.

KEEP THIS AND OTHER MEDICATIONS OUT OF THE REACH OF CHILDREN.

DIRECTIONS FOR USE

Unless otherwise directed by a prescribing healthcare practitioner, Neocera Cream should be applied to affected area three times a day (or as needed). Neocera Cream should be rubbed into the skin until it is completely absorbed.

Cleanse affected area with a mild or soap-free cleanser before applying Neocera Cream.

Non-astringent moisturizers may be used if necessary.

HOW SUPPLIED

Neocera Cream is supplied in a 450 gram tub with pump top bearing the Item Code 23710-210-45, a 3 gram professional sample plastic tube (23710-210-03) and a 450 gram professional sample tub with pump top (23710-210-50).

Store at controlled room temperature 15° to 25°C (59° to 77°F).

Device is stable up to 45 days after opening.

Neocera Cream is manufactured in the USA for
Exeltis USA Dermatology, LLC
Florham Park, NJ 07932
(877) 324-9349



www.ExeltisUSA.com

Rev. 09/2017 2100301-03