

Recedo Topical Gel

Patient Information and How to Use

RECEDO[®]
TOPICAL GEL

Because what remains, matters.

What is Recedo topical gel?

Recedo topical gel is a flexible, waterproof, and breathable silicone treatment used topically (on the skin) to treat old and new scars resulting from burns, general surgical procedures, or trauma wounds.

Recedo topical gel is available by prescription only and is intended for use only under the supervision of a healthcare practitioner.

What is Recedo topical gel used for?

Recedo topical gel is used to flatten, soften, and smooth scars, relieve the itching and discomfort of scars, as well as reduce the discoloration associated with scars.

Important information:

- Do not use Recedo topical gel if you have a known hypersensitivity to any of the gel ingredients.
- Avoid direct contact with clothing until gel is dry. Wet product may stain clothing.

How should I use Recedo topical gel?

1. **Prime pump before first use** to start product flow; priming may take up to 8 pumps.
2. **Ensure the affected area is clean and dry.**
3. Unless otherwise directed, **apply Recedo evenly in a thin layer twice daily.** Recommended **minimum treatment time is 60–90 days.** Treatment for older and larger scars may take longer than 90 days to achieve full benefit. Talk to your provider if extended use is recommended. **For maximum effect, Recedo should have 24-hour contact with the skin.**
4. **If Recedo has not dried within 4-5 minutes, you may have applied too much.** Gently remove excess and allow the drying process to continue.
5. Once dry, **Recedo can be covered with sunscreen, cosmetics or pressure garments.**

What should I avoid while using Recedo topical gel?

Recedo topical gel is intended for external use only; avoid direct contact with eyes, mucous membranes, third degree burns, and open wounds. If irritation occurs, discontinue use and consult your physician.

How should I store Recedo topical gel?

Recedo topical gel should be stored below 25°C (77°F).

Keep Recedo topical gel and all medicines out of reach of children.

Important Safety Information

Rx Only – Prescription Medical Device – Caution: Federal Law restricts this product to sale by, or on the order of a licensed healthcare practitioner.



Recedo Topical Gel

For Topical Dermatological Use Only



Because what remains, matters.

Important Safety Information (cont'd)

Description

Recedo® Topical Gel is an advanced silicone treatment for minimizing the appearance of scars. The gel dries rapidly to form an invisible protective silicone sheet over the affected area. This protective barrier is flexible, waterproof, and breathable. The patented silicone gel formulation has been shown to flatten, soften, and smooth scars, relieve the itching and discomfort of scars, as well as reduce the discoloration associated with scars.



Indications for use

Under the supervision of a healthcare practitioner, Recedo® Topical Gel is intended for the management of old and new hypertrophic or keloid scarring resulting from burns, general surgical procedures or trauma wounds.

Contraindications

Known hypersensitivity to any of the gel ingredients.

Precautions

Avoid direct contact with clothing until the gel is dry. Wet product may stain clothing.

Warnings

For external use only; avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. If irritation occurs, discontinue use and consult your physician. Keep out of the reach of children. Store below 25°C (77°F).

Directions for use

Ensure the affected area is clean and dry. Unless otherwise directed by your healthcare practitioner, apply Recedo evenly in a thin layer twice daily. For maximum effect, Recedo should have 24-hour contact with the skin. Once dry, Recedo can be covered with sunscreen, cosmetics or pressure garments. If Recedo has not dried within 4-5 minutes, you have probably applied too much. Gently remove the excess and allow the drying process to continue.

Recommended minimum treatment time is 60–90 days. It may take longer than 90 days for older and larger scars to achieve full benefit. Talk to your physician if extended use recommended.

Ingredients

Polysiloxanes, Silicon Dioxide

Please see Full Prescribing Information.